

# POST-CURING VALUE FOR VALIDATED RESINS (WHIP MIX IS NOT A RESELLER)



KEYSTONE RESINS	T = TIME (MINUTES)	P = INTENSITY	D = DUTY CYCLE	B = BOTTOM LIGHT
KEYMASK, KEYMODEL ULTRA, KEYORTHO MODEL, KEYMODEL, KEYORTHO IBT, KEYSPLINT SOFT, KEYTRAY		T = 10, P = 16, D = 10, B = ON		
KEYGUIDE		T = 5, P = 16, D = 10, B = ON		
KEYSPLINT HARD		T = 15, P = 10, D = 5, B = ON		



DENTONA RESINS	T = TIME (MINUTES)	P = INTENSITY	D = DUTY CYCLE	B = BOTTOM LIGHT
GUIDE, IBT, MODEL GB, MODEL IVORY, MODEL GREY, ORTHO MODEL, IBT, SPLINT, TEMP, MATCH, MOCK UP		T = 10, P = 13, D = 8, B = ON		
CAST DENTURE		T = 5, P = 13, D = 8, B = ON		
TRAY		T = 7, P = 13, D = 8, B = ON		
GINGIVA		T = 20, P = 13, D = 8, B = ON		



SHERA RESINS	T = TIME (MINUTES)	P = INTENSITY	D = DUTY CYCLE	B = BOTTOM LIGHT
CAST & TRAY		T = 5, P = 13, D = 8, B = ON		
TRAY		T = 7, P = 13, D = 8, B = ON		
GINGIVA, MODEL PLUS UV GREY, MODEL PLUS UV SAND, MODEL PLUS UV SMOKY WHITE		T = 10, P = 13, D = 8, B = ON		