

## ARMOR SHINE & MODEL GLOW INSTRUCTIONS FOR USE

**Armor Shine:** Brush on evenly to desired areas. Dries quickly to a clear hard finish. Additional coats may be applied to achieve a higher gloss. Water-soluble. Rinse brush after use.  
One coat adds less than one (1) micron of thickness.

**Model Glow:** May be used full strength or as a 50% dilution. To use in diluted form, mix one (1) part water to one (1) part Model Glow soap. Before soaping, make sure models are thoroughly dry. Submerge dry models completely into soap for 15 to 30 minutes. Remove models and rinse thoroughly under tap water. Models may be buffed immediately (wet) or allow models to dry then buff with a soft dry cloth until a high luster appears.

For increased longevity, keep soap in a closed container. Change soap every 6-8 weeks or when solution becomes cloudy.

**CAUTION:** Keep out of the reach of children. Avoid contact with eyes. Discontinue use if irritation occurs.