SURGICAL GUIDE

For **Carbon** printers

3D PRINT RESIN
INSTRUCTIONS

3D-DRUCKHARZ
ANLEITUNG

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INDICATION

Whip Mix Surgical Guide 3D Print resin is intended for the fabrication of drilling guides used in dental surgery. The Surgical Guide formulation is optimized for the requirements of robust production, guaranteeing consistent high quality. Surgical Guide resin was successfully tested for biocompatibility and meets all mechanical and application demands when parts are sufficiently post-cured.

CONTRAINDICATION

The material should not be used for any other purpose than the additive production of the specified dental parts. Do not use the polymerized material if allergies exist against any of the ingredients or if contact allergies exist.

PROCESSING

- Surgical Guide resin bottles should be swiveled several times (not shaken) before use.
- Make sure that Surgical Guide resin is stored 18°C to 28°C and utilized between 23°C to 40°C.
- Carefully pour Surgical Guide resin into the cassette.
- After the build process is finished, a direct post treatment is recommended immediately.
- Clean parts with fresh isopropanol alcohol, ≥97% in an ultrasonic bath and air dry for 10 – 15 minutes prior to UV post-cure.
- Place objects in an adequate UV curing unit Based on your UV curing unit, follow the relevant instructions below:
 - Otofash G171: 2x 2000 flashes (turn components around after 2000 flashes)
- Support removal is recommended after UV post-cure.
- Impurities due to operation mistakes cannot be excluded. With respect to the low viscosity of the resin, it is possible to filter Surgical Guide resin. Recommended to filter size diameter <100µm.
- Surgical guide can be disinfected.

COMPOSITION

- Mixture of acrylate and methacrylate resins, photoinitiators (phosphine oxide), additives, dyes.

IMPORTANT

- To avoid detrimental effects on material quality, do not expose the liquid to radiation under any circumstances.
- Deviations from the described manufacturing process may lead to different mechanical and optical properties of the Surgical Guide resin.
- Optimal layer thickness is 50µm.
- Ensure personal protective gear during processing.
- Polymerized resins are chemically resistant – avoid stains on clothing.
- Avoid any contact with skin and eyes. In case of accidental contact, rinse with adequate running water. Consult a doctor if necessary.
- Use by qualified personnel only.
- Patient target group of the medical devices: adults and adolescents
- Keep out of the reach of children!
- The lot number and best before date are indicated on each Surgical Guide resin package. In case of claims, please always indicate the lot number of the product. Do not use the product after expiration of the best before date.
- Report all serious incidents (death, serious deterioration of health, serious risk to public health) that have occurred in connection with the medical device to the manufacturer and the competent authority of the member state.

PHYSICAL PROPERTIES FOR 385nm:

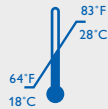
Product description:

Photopolymerizable resin for production of drilling guides.

Technical data:

- **Color:** Translucent
- **Density:** ca. 1.1 g/ml
- **Viscosity:** ca. 0.7 Pa s
- **Post cured material:**
with Otofash G171
- Elastic Modulus:** ca. ≥ 1600 MPa
- Flexural Strength:** ca. ≥ 60 MPa
- Hardness:** ca. 75 ($\pm 10\%$) Shore D

Storage:

Ordering information:

Standard packing: 1000g (910ml)

REF 71079-C

Translucent

WARNING!

HAZARD STATEMENTS

Causes skin irritation.
May cause an allergic skin reaction.
Causes serious eye irritation.
May cause respiratory irritation.
Harmful to aquatic life with long lasting effects.

PRECAUTIONARY STATEMENTS:

Avoid breathing mist, vapors or spray.
Wash thoroughly after handling.
Use only outdoors or in a well-ventilated area.
Contaminated work clothing must not be allowed out of the workplace.
Avoid release in the environment.
Wear protective gloves and eye protection.

IF ON SKIN: Wash with plenty of soap and water.

If skin irritation or rash occurs: Get medical attention.

Take off contaminated clothing and wash before reuse.

IF IN EYES: Rinse cautiously with water for several minutes.

Remove contact lenses, if present and easy to do. Continue rinsing.

If eye irritation persists: Get medical attention.

IF INHALED: Move person to fresh air and keep comfortable for breathing.

Call a POISON CENTER or doctor if you feel unwell.

Store in a well-ventilated place. Keep container tightly closed.

Store locked up.

Dispose of contents and container in accordance with local and national regulations.

The information in this document includes typical values from printing various parts and is intended for reference and comparison purposes only. This information should not be used for testing, design specification, or quality control purposes. End-use material performance can be impacted by, but not limited to, design, processing, operating and end-use conditions, test conditions, color, etc. Actual values will vary with build conditions. In addition, product specifications are subject to change without notice.

This information, along with any technical advice from Whip Mix and/or its Third-Party Partners, is given to you in good faith but without warranty. The application, use, and processing of these and other Whip Mix products by you are beyond the control of Whip Mix and its Third-Party Partners and, therefore, entirely your own responsibility. Whip Mix products are only to be used by you subject to the terms of any written agreement(s) by and between you and Whip Mix and/or its Third-Party Partners.

You are responsible for determining that the material in the Whip Mix products is safe, lawful, and technically suitable for the intended application, as well as for identifying the proper disposal (or recycling) method consistent with applicable environmental laws and regulations. WHIP MIX AND ITS THIRD-PARTY PARTNERS MAKE NO WARRANTIES OF ANY KIND, EXPRESS OR IMPLIED, INCLUDING, BUT NOT LIMITED TO, THE WARRANTIES OF MERCHANTABILITY, FITNESS FOR A PARTICULAR USE, OR NON-INFRINGEMENT. Further, it is expressly understood and agreed that you assume and hereby expressly release Whip Mix and its Third-Party Partners from all liability, in tort, contract or otherwise, incurred in connection with the use of Whip Mix products and any associated information or technical advice. Any statement or recommendation not contained herein is unauthorized and shall not bind Whip Mix or its Third-Party Partners. Nothing herein shall be construed as a recommendation to use any product in conflict with any claim of any patent relative to any material or its use. No license is implied or granted under the claims of any patent.

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Whip Mix Corporation ■ 361 Farmington Avenue ■ Louisville, KY USA 40209