VERITRAY™

GENERATIVE RESIN
INSTRUCTIONS

GENERATIVES HARZ
ANLEITUNG

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RESINA GENERADORA
INSTRUCCIONES

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PRODUCT DESCRIPTION

Whip Mix VeriTray™ is a generative resin for the production of customized dental impression trays based on a image projection system (≤ 385 nm). The formulation of VeriTray is optimized for the requirements of a robust production guaranteeing constant high quality. VeriTray is successfully tested for biocompatibility, certainly meets all mechanical and application demands. The material can be used for build processes with layer thicknesses from 50 up to 100 μm . Use of the VeriCure™ curing device is recommended for post curing.



PROCESSING

- VeriTray bottles should be well shaken before use (fig 1).
- Make sure that VeriTray material is at a temperature from 23°C to 30°C.
- Carefully pour VeriTray into the vat of the image projection unit (fig 2).
- Bubbles can be removed with a cleaned spatula or by a recoater routine.
- If possible, always store a bottle of VeriTray in you production unit to avoid temperature differences during refilling.
- For the machine parameter adjustment please refer to the machine data sheet.
- After the build process is finished a direct post treatment is recommended. If this cannot be guaranteed leave the produced objects in the liquid VeriTray resin.
- After cleaning of the parts with isopropanole $\geq 97\%$ (approx. 3 – 5 minutes in an ultrasonic bath) the objects are postcured in an adequate light curing unit (e.g. VeriCure for a period of 3 – 4 min) in a protective gas atmosphere.
- The dental objects generated out of the VeriTray generative resin can be coated and repaired as usual. For this purpose Whip Mix products e.g. L-1 UV lacquer are recommended.
- Impurities due to operation mistakes cannot be excluded. With respect to the low viscosity of the resin it is possible to filter VeriTray resin. It is recommended to filter and stir up the resin regularly. If opaque material is used additionally, mix the content with a propeller mixer carefully (fig. 3). To avoid bubbles let VeriTray rest for 30 minutes before usage.

Contains: Urethanedimethacrylates, initiators, dyes, and stabilisers.

IMPORTANT

- To avoid detrimental effects on material quality do not expose the liquid material to irradiation under any circumstances.
- Deviations from the described manufacturing process may lead to different mechanical and optical properties of the VeriTray material.
- Ensure personal protective gear during processing.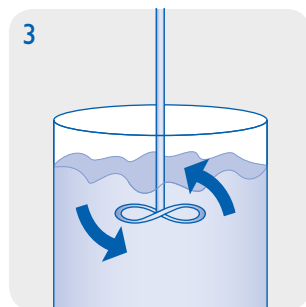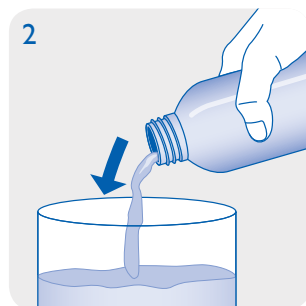

- **Caution:** Polymerised resins are chemically resistant – avoid stains on clothing!
- Avoid any contact with skin and eyes. In case of accidental contact, rinse with adequate running water, consulting a doctor if necessary.
- The lot number and the best before date are indicated on each VeriTray package. In case of claims please always indicate the lot number of the product. Do not use the product after expiration of the best before date.

SAFETY ADVICE


Whip Mix Corporation is not liable for any damages caused by improper application of the material. To be used by trained specialist personnel for the purpose indicated only.

PRODUCT DESCRIPTION:

Photopolymerizable resin for production of customized impression trays by image projection systems (≤ 385 nm)

TECHNICAL DATA:

- **Color:** Orange, Neon Green
- **Density:** ca. 1.1 g/ml
- **Viscosity:** ca. 0,8 Pa s
- **Green flex modulus:**
Elastic modulus: ca. 600 MPa
Flexual strength: ca. 50 MPa
Elongation at break: ca. 20%
- **Post cured material:**
 (depends on postcuring unit)
Elastic modulus: ca. 1750 MPa
Flexual strength: ca. 78 MPa
Elongation at break: ca. 11 %
Hardness: ca. 80 Shore D
- **Storage:**

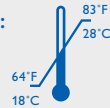ORDERING INFORMATION:

Standard packing: 1000g (910ml),
 item no.: **71082 Orange;**
71084 Neon Green

WARNING!

Hazard statements

Causes skin irritation.
 May cause an allergic skin reaction.
 Causes serious eye irritation.
 May cause respiratory irritation.
 Harmful to aquatic life with long lasting effects.

Precautionary statements:

Avoid breathing mist, vapors or spray.
 Wash thoroughly after handling.
 Use only outdoors or in a well-ventilated area.
 Contaminated work clothing must not be allowed out of the workplace.
 Avoid release to the environment.
 Wear protective gloves and eye protection.
 IF ON SKIN: Wash with plenty of soap and water.

If skin irritation or rash occurs: Get medical attention.
 Take off contaminated clothing and wash it before reuse.
 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
 If eye irritation persists: Get medical attention.
 IF INHALED: Remove person to fresh air and keep comfortable for breathing.
 Call a POISON CENTER or doctor if you feel unwell.
 Store in a well-ventilated place. Keep container tightly closed.
 Store locked up.
 Dispose of contents and container in accordance with local and national regulations.

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Whip Mix Corporation ■ 361 Farmington Avenue ■ Louisville, KY USA 40209