

magnetic SYSTEM CONVERTER



For Denar®, Hanau™ and Whip Mix articulators.
Call your sales rep for details.



RESTORATIVE ORAL HEALTH

Whip Mix Corporation
361 Farmington Avenue
Louisville, KY USA 40209
To order, call your Whip Mix Customer Service Representative
at 1-800-626-5651 or your dental dealer today.
www.whipmix.com

© 2017 Whip Mix Corporation

FN 20002371-F-AE R0617

Magnetic retention
without adding steps
to the process.

Whip Mix®, Denar® and logos are registered trademarks and Hanau™ is a trademark of Whip Mix Corporation.


ACCURATE

REPEATABLE

FLEXIBLE

STRONG

QUICK

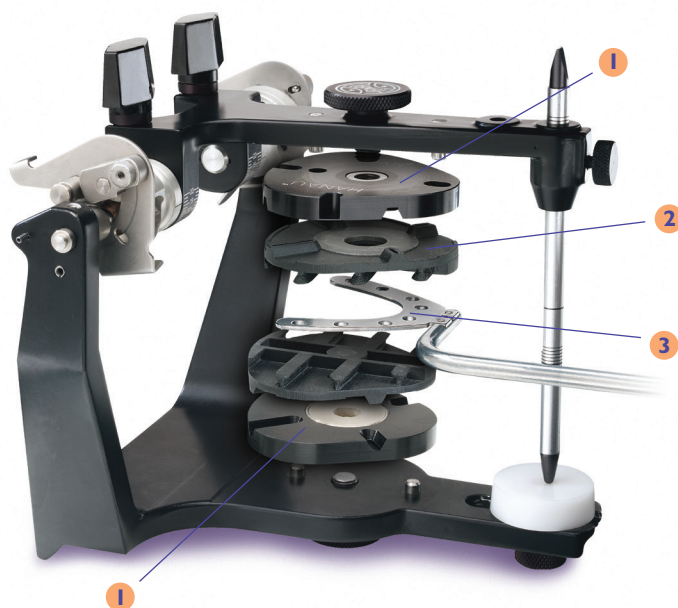

instructions

- 1 Attach converter plate to upper and lower members just like you would a traditional mounting plate.
- 2 Snap disposable mounting plate onto converter plate. (same for upper and lower)
- 3 Locate maxillary cast with a facebow or mounting platform.
- 4 Close the articulator and mount the cast with stone to the disposable mounting plate.

NOTE: The stone must harden completely before removing the mounting plates from the converter. If you choose to hand finish the plaster, work with the model while on the articulator, or unscrew the converter plate.

magnetic SYSTEM CONVERTER