

Facebow Level

To aid in ensuring that the facebow is mounted on the patient's head parallel to the horizon, two peel and stick levels are included in your case.

To verify the facebow is level side-to-side, peel the tape off the back of the level and attach the level to a flat surface on the front of your facebow. Most commonly the level is mounted on top of the sight in the center of the facebow as pictured.

You can either use the anterior reference pointer or the second level to help ensure the facebow is level front-to-back. If you choose to attach the second level, it is most commonly placed on top of the left facebow arm setting parallel near the Denar logo (option B).

Do not autoclave the facebow with level.
To sterilize, wipe with disinfectant wipe.

Additional levels can be ordered using item number 20002951.



Option B



RESTORATIVE ORAL HEALTH

Whip Mix Corporation
361 Farmington Avenue
Louisville, KY 40209
Toll-Free 800-626-5651
Fax: 502-634-4512

www.whipmix.com

FN: 20002952-F AA 0210