

Congratulations on the Purchase of Your Bead Dryer Belt!

The Whip Mix Bead Dryer Belt warms blasting abrasives to prevent clumping due to moisture and insure that the beads flow through the strainer and transport reliably.

To use, place the belt around the middle of the bead container and stretch springs to connect. Plug into a 110V wall socket and keep plugged in until beads are gone. May be re-used indefinitely. The product is equally effective with glass beads and aluminum oxide abrasive in the Whip Mix Xcavator or any conventional manual microblaster.



Bead Dryer Belt



Belt wrapped around 50lb.
Bead Bucket

