

Quick Start Guide for ProFox Oximetry Software Distributed by Whip Mix

1

Open ProFox Oximetry

Main Menu

- New Patient
- Select Another Patient
- Oximetry Data (transfer)
- Print or Preview Reports
 - Make sure a printer is installed before clicking on this option

2

Connect Pulse Oximeter

- Detach finger probe
- Connect USB Cable
- Plug USB cable into computer



3

Select Oximetry Data (transfer)

- Select Nonin 3150 WristOx2
- Select Com Port for USB connection
- Highlight session you want to save, click ok
- Graph will be displayed to save, click on save this test
- Verify data belongs to correct patient
 - If wrong patient, click no to setup new patient
 - If yes, click yes to add specific test data to comments
- Edit or print test from main menu

4

To View All Saved Tests

- Select Another Patient
- Select first letter of last name for patient search

5

Update Data in Oximeter

- Utilities
 - Oximeter Communications Check
 - Set Time
 - Erase Memory
 - You may want to erase memory before each new patient test
 - Make sure all tests are saved before erasing memory
- Keep Oximetry Log
 - Write down date and time patient wears oximeter

Distributed by:
Whip Mix Corporation
Louisville, KY 40209
800-626-5651
502-637-1451
502-634-4515 (fax)
www.whipmix.com
01/2019