



RESTORATIVE ORAL HEALTH

QuickMount Magnetic System™

A universal magnetic mounting system for semi-adjustable articulators

The QuickMount Magnetic System allows the quick, precise and easy removal and replacement of mounted casts on several of the most popular semi-adjustable articulators. The end result is the saving of time and money.

The upper and lower members of the selected semi-adjustable articulator are fitted with the QuickMount Plate Holder which is used in place of the regular mounting plate screws and/or knobs. The QuickMount Plates are made of a highly-filled composite resin material and are held precisely and firmly in place by the mounting plate pins and the super magnet. Each QuickMount Plate can be easily removed by simply breaking the magnetic contact. This eliminates the tedious and time-consuming process of screwing the plates on and off.

The mounting plate is designed for use with Whip Mix, Panadent or Hanau 2 Pin articulators.

Articulator Preparation Prior to Attaching QuickMount Plate Holders:

WHIP MIX / PANADENT Articulators

1. Using a 3/32 inch Allen wrench, remove the set screws from upper and lower member of the articulator. Some set screws may be concealed with caulking that's easily removed with a hand instrument.
2. Remove the mounting plate thumbscrew assemblies and store for future use.
3. Enlarge the mounting plate screw holes to 5.0mm diameter, if necessary.

HANAU 2 Pin Articulators

1. Unscrew the mounting plate thumbscrews from the upper and lower members of the articulator.
2. Store the thumbscrew assemblies for future use.

Attachment of QuickMount System to the Articulator:


1. Position a QuickMount Plate, without the metal disk in place, onto the mounting plate pins of the articulator.
2. Place a QuickMount Plate Holder with threaded end inserted through the articulator frame and attach the QuickMount Plate Holder Nut to the threaded end of the QuickMount Plate Holder and tighten lightly.
3. Hold QuickMount Plate Holder anteriorly and tighten the QuickMount Plate Holder Nut firmly with wrench.
4. Remove QuickMount Plate, place a metal mounting disk into the QuickMount Plate and reattach to the articulator.

Item No.

28701 QuickMount Plates with metal disks (Pkg. of 20) for:
Whip Mix, Panadent, Hanau 2 Pin Articulators

28703 QuickMount Plate Holder 13mm for semi-adjustable articulators except the following: Lower frame of Whip Mix Accumount Articulator Model #8340, #2240, #3040, #3140 (Includes plastic wrench)

28704 QuickMount Plate Holder 25mm for lower frame of Whip Mix Accumount Articulator Models #8340, #2240, #3040, #3140

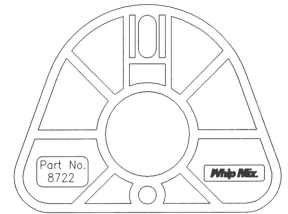

Selection Guide

Item No.	28701	28703	28704
	Plates (Pkg. 20)	13mm Plate Holder	25mm Plate Holder
Whip Mix: 8500, 8300, 2200, DB2000	X	2	
Whip Mix: 8340, 2240, 2340, 3040, 3140	X	1	1
Panadent	X	2	
Hanau (Modular)	X	2	
Hanau (Wide Vue)	X	1	1

