

REMOUNTING RECORD JIG

THE REMOUNTING RECORD JIG is employed to:

- (1) Avoid the necessity for repeating facebow transfers for correcting occlusion.
- (2) Provide a permanent record for remounting full or partial dentures on the articulator in their original relationship to the condyle.

thus releasing the articulator mounting rings for other cases in progress.

The Remounting Record Jig is attached to the lower member of the articulator, and a smooth layer of quick-setting plaster about 3/4" thick (or as necessary to obtain the desired matrix) is placed on the keyed platform. Close the articulator so that the occlusal surface and incisal edges of the teeth register in the soft plaster. The plaster remounting record can be filed with the patient's name, date, and articulator serial number and calibration readings written on the keyed side.

NOTE: Height of this remounting jig can be reduced by removing one or two sets of spacers held by bottom screws.



RESTORATIVE ORAL HEALTH

Whip Mix Corporation - West
1730 East Prospect Rd., Suite 101
Fort Collins, CO 80525
Toll-Free: 1-800-201-7286
Fax: 1-970-472-1793
www.whipmix.com

Whip Mix® is a registered trademark of
Whip Mix Corporation

©2008 Whip Mix Corporation

FN 339469-F AC R1108