



RESTORATIVE ORAL HEALTH

READ THIS FIRST!

In order to ensure accurate results with Best-Bite, please take 10 minutes to do the following:

- Watch the two brief videos on the enclosed CD (plays on any PC or MAC computer)
- Read the enclosed instruction packet
- Patients must tap teeth when wearing the Discluder
- Practice your first fabrication on a staff member
- Always use safety leash after initial fabrication
- Always have patients sign the patient release form
- Lack of pain relief indicates the muscles are not the primary source of pain
- Pain relief confirms the role of muscles as a potential source of pain
- Best-Bite should be used by the patient as a rescue device at home during treatment

If you have questions, please contact:
Whip Mix Customer Service at 800-626-5651.