VERIMODEL™

GENERATIVE RESIN
INSTRUCTIONS

GENERATIVES HARZ
ANLEITUNG

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RESINA GENERADORA
INSTRUCCIONES

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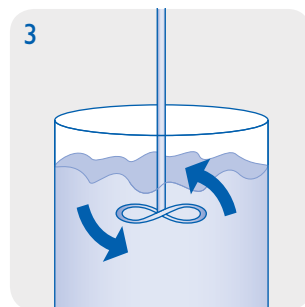
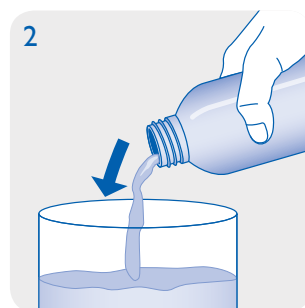
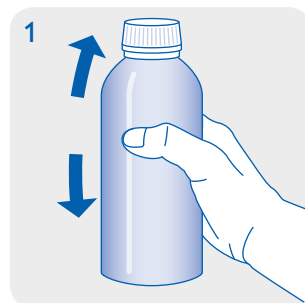
PRODUCT DESCRIPTION

Whip Mix VeriModel™ is a generative resin for the production of dental models based on a image projection system (≤ 385 nm). The formulation of VeriModel is optimized for the requirements of a robust production guaranteeing constant high quality. The material can be used for build processes with layer thicknesses from 10 up to 100 μm . Use of the VeriCure™ curing device is recommended for post curing.

PROCESSING

- VeriModel bottles should be well shaken before use (fig 1).
- Make sure that VeriModel material is at a temperature from 23°C to 30°C.
- Carefully pour VeriModel into the vat of the image projection unit (fig 2).
- Bubbles can be removed with a cleaned spatula or by a recoater routine.
- For the machine parameter adjustment please refer to the Technical Data Sheet.
- After the build process is finished a direct post treatment is recommended. If this cannot be guaranteed leave the produced objects in the liquid VeriModel resin.
- After cleaning of the parts with isopropanole $\geq 97\%$ (approx. 3 – 5 minutes in an ultrasonic bath) the objects are postcured in an adequate light curing unit (e.g. VeriCure for a period of 3 – 4 min) in a protective gas atmosphere.
- Dental models generated out of the VeriModel generative resin can be coated and repaired as usual. For this purpose Whip Mix products e.g. L-1 UV lacquer are recommended.
- Impurities due to operation mistakes cannot be excluded. With respect to the low viscosity of the resin it is possible to filter VeriModel resin. It is recommended to filter and stir up the resin regularly. If opaque material is used additionally, mix the content with a propeller mixer carefully (fig. 3). To avoid bubbles let VeriModel rest for 30 minutes before usage.

Contains: Alkoxilated bisphenole-A-dimethacrylate, initiators, dyes, stabilisers, and pigments.

IMPORTANT

- To avoid detrimental effects on material quality do not expose the liquid material to irradiation under any circumstances.
- Deviations from the described manufacturing process may lead to different mechanical and optical properties of the VeriModel material.
- Ensure personal protective gear during processing.
- **Caution:** Polymerised resins are chemically resistant – avoid stains on clothing!
- Avoid any contact with skin and eyes. In case of accidental contact, rinse with adequate running water, consulting a doctor if necessary.

- The lot number and the best before date are indicated on each VeriModel package. In case of claims please always indicate the lot number of the product. Do not use the product after expiration of the best before date.

SAFETY ADVICE

Whip Mix Corporation is not liable for any damages caused by improper application of the material. To be used by trained specialist personnel for the purpose indicated only.

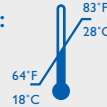
This product contains bisphenol A which is known to the State of California to cause birth defects or other reproductive harm.

PRODUCT DESCRIPTION:

Photopolymerizable resin for production of dental models

TECHNICAL DATA:

- **Color:** Golden Brown, Violet, Blue, Yellow, Ivory, Green or White
- **Density:** ca. 1.1 g/ml
- **Viscosity:** ca. 0,7 Pa s
- **Post cured material:** (depends on postcuring unit)
- **Elastic modulus:** ca. 1700 MPa
- **Flexural strength:** ca. 90 MPa
- **Elongation at break:** ca. 10 %
- **Hardness:** ca. 80 Shore D
- **Storage:**

ORDERING INFORMATION:

Standard packing: 1000g (910ml),
item no.: **71081 Golden Brown;**
71083 Violet; 71085 Blue;
71086 Yellow; 71087 Ivory;
71088 Green; 71089 White

WARNING!

Hazard statements

Causes skin irritation.
May cause an allergic skin reaction.
Causes serious eye irritation.
May cause respiratory irritation.
Harmful to aquatic life with long lasting effects.

Precautionary statements:

Avoid breathing mist, vapors or spray.
Wash thoroughly after handling.
Use only outdoors or in a well-ventilated area.
Contaminated work clothing must not be allowed out of the workplace.
Avoid release to the environment.
Wear protective gloves and eye protection.
IF ON SKIN: Wash with plenty of soap and water.

If skin irritation or rash occurs: Get medical attention.
Take off contaminated clothing and wash it before reuse.
IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

If eye irritation persists: Get medical attention.
IF INHALED: Remove person to fresh air and keep comfortable for breathing.
Call a POISON CENTER or doctor if you feel unwell.
Store in a well-ventilated place. Keep container tightly closed.

Store locked up.
Dispose of contents and container in accordance with local and national regulations.

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Whip Mix Corporation ■ 361 Farmington Avenue ■ Louisville, KY USA 40209