


AquaSpense Quick Start Guide

Quick Set Up




- 1 Remove the AquaSpense from the packing material and place it on a flat surface.
- 2 Attach the scale cord to the scale socket on the back of the AquaSpense.
- 3 Install the power cord and plug into a grounded outlet.
- 4 Fill the water bottle with distilled water, then screw bottle on the cap by turning the bottle and place on the bottle holder.


Add a Program

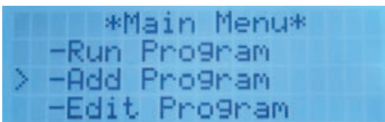


- 1 Turn on the AquaSpense. **Main Menu** will be on the screen. Use the arrow \blacktriangle \blacktriangledown buttons to select **Add Program** then press Enter button.

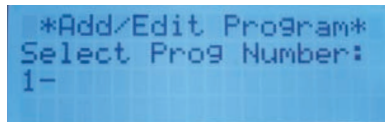

- 2 Use the \blacktriangle \blacktriangledown buttons to select the desired program number, then press Enter button.

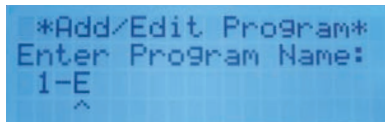


- 3 Use the \blacktriangle \blacktriangledown buttons to select numbers and letters of the program name, then press \blacktriangleright button to move to the next position. Press \blacktriangleleft button to move back.

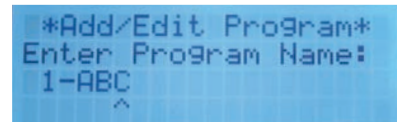


- 4 Once the name is complete, press Enter button to move to the next screen.



- 5 Use the buttons \blacktriangle \blacktriangledown to input powder weight for the powder/water ratio (usually 100g). Use \blacktriangleleft \blacktriangleright buttons to advance or move back. Press Enter button to move to the next screen.

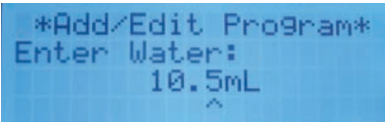


- 6 Use the buttons \blacktriangle \blacktriangledown to indicate the amount of water to dispense per the powder/water ratio. Use \blacktriangleleft \blacktriangleright buttons to advance or move back. Press Enter button to save the program.

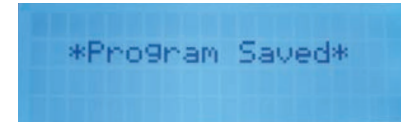


- 7 The program is saved. *Although you entered the information for one powder weight, the unit will dispense correctly for any weight.*

Run a Program

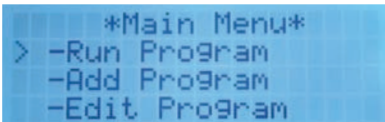

- 1 Select Run Program then press Enter button.

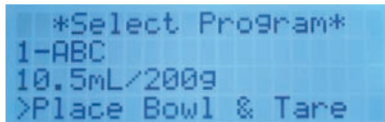


- 2 Use the \blacktriangle \blacktriangledown buttons to select programs you wish to run.



- 3 Place the empty bowl on the scale.




- 4 Press the Tare button.

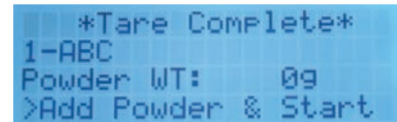

- 5 When the Tare is complete the unit will indicate this on the display and it will beep. Do not remove bowl until Tare is complete.




- 6 Remove the bowl and add desired powder.

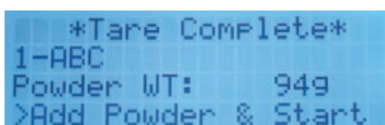


- 7 The display will indicate the amount of powder in the bowl.



- 8 Press the Start button to run the program.

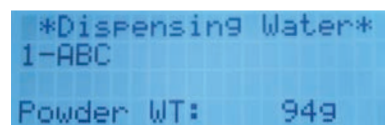


- 9 The display will indicate when AquaSpense is dispensing water.

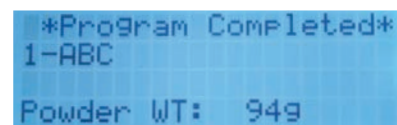

- 10 The unit will beep and the display will indicate when the program is complete and the batch is ready.

Tips

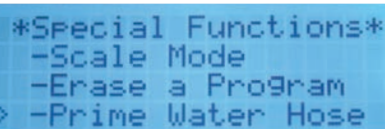

Use the extension hose to draw water from a larger container.

Prime the water hose to clear the bubbles from the hose.



more reasons. one source