

Retrofitting The QuickMount Magnetic System to Your Articulator



Completed assembly of QuickMount Magnetic System with Mounting Plate on Model #2240 Articulator

Step 1: Removing Knob



Remove sealing compound covering Hex Screw



Using Hex Driver, loosen screw



Knob should now remove easily

Step 2: Attaching Magnetic Plate Holder



Position Magnetic Plate Holder on Upper Member



Attach QuickMount Plate Holder Nut to threaded post



Lightly tighten QuickMount Plate Holder Nut and repeat steps for Lower Member

Step 3: Positioning and Securing Magnetic Plate Holder



Place the QuickMount Mounting Plate over Magnet



Push Magnet forward against QuickMount Mounting Plate



Place Blue Wrench on the QuickMount Plate Holder Nut



With Magnet in the correct position, tighten Nut.

Use care not to overtighten nut

Step 4: Attaching Mounting Plate



Remove QuickMount Plastic Mounting Plate and place Metal Disk over Slot



Press the Metal QuickMount Disk into slot on Plastic Mounting Plate



Attach the QuickMount Plastic Mounting Plate onto articulator



Correct assembly of QuickMount Mounting plate on Model #2240 Articulator